

SOMATOM go.Now Compact

Run a successful CT business

www.siemens-healthineers.co.in



AI Powered

96
Slice
CT



SOMATOM go. platform


How we got here
It all started with you, our customers...

...and the actual challenges you face
every day.

The
healthcare
market is
changing



To address
these changes,
we started our
**biggest CT
innovation
project** ever in
co-creation

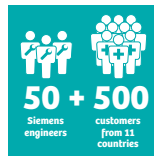

The result is
a whole new
**SOMATOM go.
platform** for
daily business
success



The SOMATOM go.Now Compact addresses the needs of routine operations and beyond. It makes high-quality care accessible and helps you to run a successful CT business – allowing you to keep an eye on profitability, stay competitive, and avoid unnecessary cost.

“Unlike in the past, patients are very much conscious about the ‘quality’ at ‘affordable’ cost. My decision to buy and install SOMATOM go.Now Compact was purely driven by above understanding.



After working with SOMATOM go.Now Compact for a year I realized, I have made the ‘right’ choice. The Image quality is excellent surpassing my expectations. The throughput and workflow are very good, all this comes with very less radiation dose and power consumption. Machine operation through tablet and visual appeal of the equipment is highly appreciable. I would strongly recommend this machine for your daily routine scanning and applications.”

Dr. P.C. Shaji


Dr. Shaji's MRI & Medical Research Centre Pvt. Ltd, Calicut

Gain flexibility with the new Mobile Workflow



Tablet

The lightweight, high-resolution tablet gives you total freedom over how you work. With Scan&GO technology, you just need a few steps for the entire scan. Start checking patient information as soon as you collect them from the waiting room, and then prepare the scan directly at the gantry to stay with the patient for longer. Since the images are sent wirelessly from the scanner to the tablet, operators can return to the patient after the scan and stay there while previewing the images and communicating with radiologists for instant feedback if required.

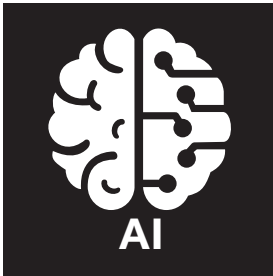

Artificial Intelligence

AI Self-learning enables protocol selection based on the clinical indications.

ALPHA technology identifies and understands human anatomy including Landmark Registration which helps in automatic planning of desired scan range.


Ease up Bolus Tracking with automatic AI FAST ROI identification and HU calculations.



Halo (Includes 2D Camera, Visual Countdown, Mood Lighting)


The gantry-integrated camera with its 90° viewing angle gives you a superb view of the tunnel on the monitor. The close-up perspective makes it easy to spot even micro-movements and keep the patient in the right position.

The ambient mood lighting and a digital visual countdown helps improve patient well-being and help them comply with breath-hold times.

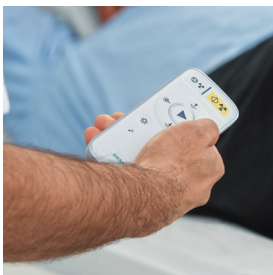

Remote Control

The easy-to-use Bluetooth remote control complements the tablet operation by streamlining scanning and making workflow processes more efficient.

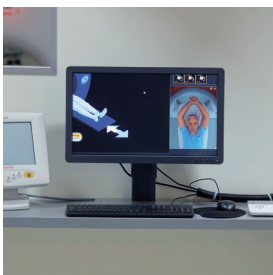

New Workplace Design

With gantry-integrated computers, SOMATOM go. platform gives you complete flexibility, more space, less noise and heating in your console room.

"There is significant ease of working with SOMATOM go.Now Compact, with all new software interface, mobile tablet and remote control. The automatic reconstruction facilitates in diagnosis saves lot of time. Angiography done is superb with this machine."



Dr. Chandrika Sahu, Sahu Diagnostics & Research Hospitals, Raipur

Automate your workflow with GO Technologies

Another important factor contributing to high performance is workflow automation. SOMATOM go.Now Compact features a holistic set of intuitive solutions that addresses your workflow not only at the scanner but also beyond. By reducing repetitive steps, GO Technologies help standardize and simplify all departmental processes – from patient setup to image distribution, archiving and reading. You can therefore work more efficiently and focus on your patients – two factors key to running a successful business.



Scan&GO

This advanced tablet app allows you to control scans remotely. You can choose whether to operate the scanner at the gantry or from outside the room to benefit from faster patient preparation and positioning.

You can also check the images quickly after the scan, as wireless connectivity sends the results to the tablet almost immediately.

Scan&GO brings an entirely new level of flexibility to your processes. Patients are also likely to feel more comfortable, since you can be with them for longer.




Check&GO

This intelligent algorithm flags up problems with coverage or contrast distribution as they occur. Correct issues on the go, prevent subsequent errors in multiphase scans, and avoid archiving suboptimal images – ultimately reducing the number of recalls.

The FAST ROI feature automatically identifies regions of interest and monitors HU for the aorta in bolus-tracking examinations.

Check&GO's automated support means that users of all levels of experience can produce high-quality images.

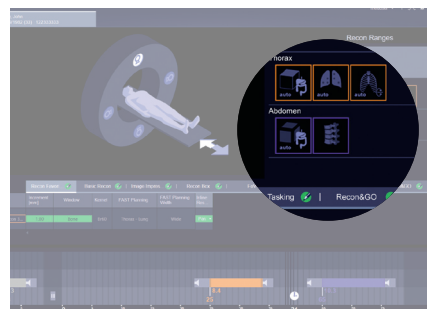

Recon&GO

Recon&GO performs zero-click postprocessing, making it part of the standard reconstruction tasks. This ready-to-read technology saves time and cuts down on workflow steps.

Recon&GO delivers high-quality results irrespective of the operator or clinical area, and allows users to spend more time with the patient.

Achieve fast, standardized, and reproducible results with this automated postprocessing and reconstruction solution.


“Thanks to Siemens Healthineers for all the support and guidance which they have rendered for tablet operated CT scan machine SOMATOM go.Now Compact. The Service Support is outstanding. Anyone can sell you the machine, but I have never seen a company so committed, making sure your system is running properly. They really go above and beyond. The radiologists and technologists are so pleased with the image resolution, quality and finish of the product. Kudos to Siemens Healthineers for having done a first-rate job.”



Dr. Roopak Chandran, Tesla Diagnostics, Angamaly, Kerala


Routine

SOMATOM go.Now Compact enables Excellent Imaging details with High resolution imaging at Lowest dose with **SAFIRE**, helps in getting Smaller details like Non-enhancing Cyst, Hemangioma's in Liver, etc.

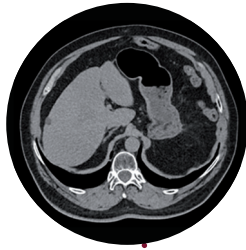


Fast and precise routine neuro examinations

SOMATOM go.Now Compact optimizes image quality and reduces dose in routine neuro examinations. High spatial resolution improves the imaging of very small bony structures such as those in the inner ear.

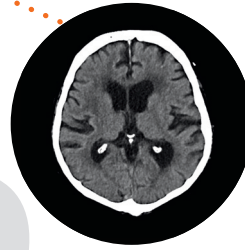

Broaden your portfolio in preventive care

SOMATOM go.Now Compact allows you to offer colon imaging and lung cancer screening. With sub-millimeter collimation in every scan, it provides high spatial resolution and helps improve the detection of polyps and ulcers.

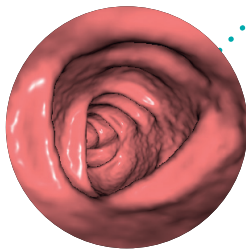

Preventive

SOMATOM go.Now Compact designed for Ultra Low dose Lung Imaging with **TIN filter** gives highest Spatial resolution and improved detection of Occult lesions e.g. Solitary pulmonary nodules.



Fast and efficient routine scanning

SOMATOM go.Now Compact delivers the low-dose scanning that is essential for optimal patient care, e.g. in orthopedics. Its technology allows scans of long ranges while maintaining a high spatial resolution.



High-quality angiography for vascular exams

SOMATOM go.Now Compact allows high-quality angiography exams of central arteries with good iodine contrast, sub-millimeter slices, and precise timing.



"It's a very user-friendly machine with excellent image quality, it is versatile and capable of doing good 3D recon, CT Angiography, Virtual Colonoscopy and Endoscopy. It is running extensively since last 10 months with very less downtime".




Dr. Anil Kumar Tyagi, Dewan Diagnostics Centre, Ghaziabad

Go for profound clinical results

Excellent detailing with Sub mm Collimation

SOMATOM go.Now Compact helps you turn complex exams into clinical routine. Thanks to premium technologies, it delivers results that were previously only available on high-end scanners.




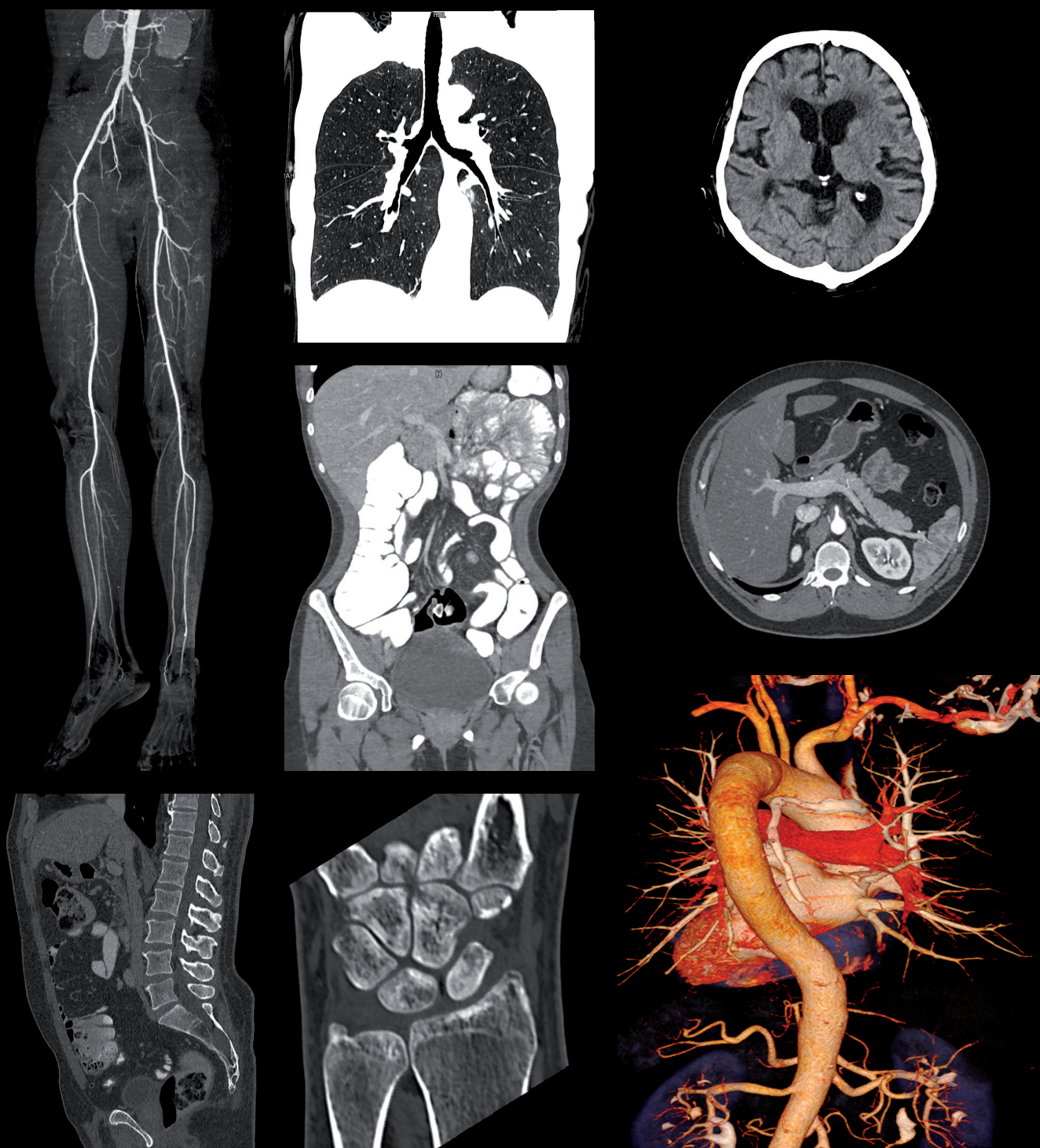


"It is real beast!

We have done all types of CT Studies without a break. Amazing built quality yet economical. I am totally satisfied with the machine. It is masterpiece from Siemens Healthineers, especially for a country like India."




Dr. Pradeep N, Sri Sai Hospitals, Bengaluru

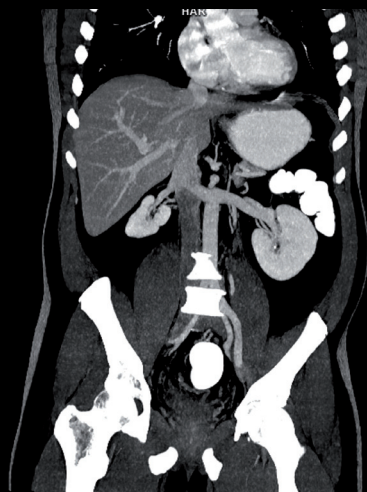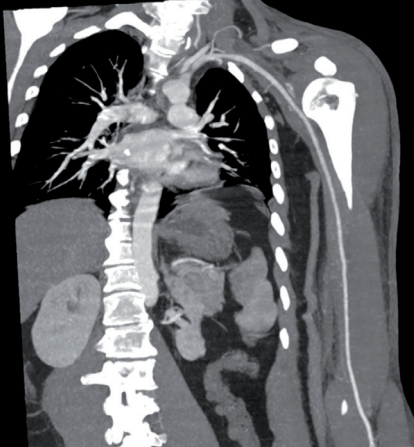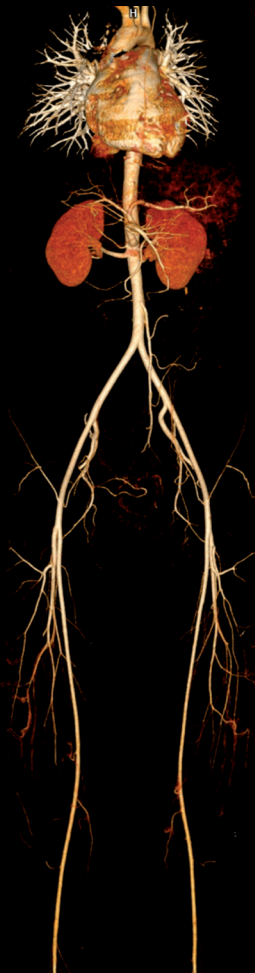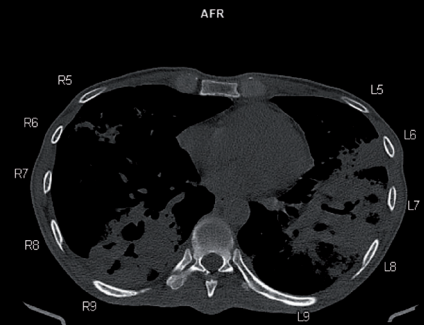

Exclusive technologies on SOMATOM go. platform

Stellar detector

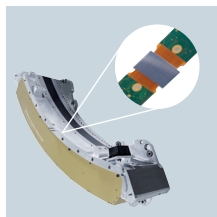

The Stellar detector lowers image noise in every scan, while advanced iterative reconstruction from SAFIRE delivers excellent image quality at very low doses. This provides excellent and homogenous image quality, even in complex areas, such as the **base of the skull**, making it especially relevant for routine neuro imaging.

The Stellar detector's high-end technology includes fully integrated components and an **advanced 3D anti-scatter collimator**. It keeps electronic noise low, increases dose efficiency, and improves spatial resolution. The smart configuration of the detector elements simplifies access, eases maintenance, and increases scanner uptime.


"Due to new Stellar detector, artifacts are significantly reduced, inspite of Magic angle and other bony artifacts especially in brain posterior fossa studies."



"Because of FAST acquisition, which is followed by automatic post processing, we can release the patients faster, especially in cases under Intubation and Ventilation on the basis of emergency."

Dr. Manojit Ghosh, Nehru Memorial Techno Global Hospital, Kolkata

Tin Filter

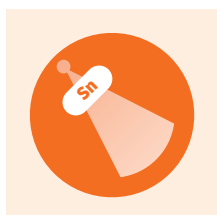

Inherited from high-end dual source scanners, the Tin Filter cuts out lower energies to reduce dose and optimize image quality at the interface between soft tissue and air. **This has direct benefits in lung, colon imaging and inner ear.**


Clinical experience also shows that **Tin Filter technology reduces beam-hardening artifacts** and improves image quality in bony structures, making it also extremely useful in orthopedic examinations.

"Apart from the compact design of SOMATOM go.Now, one feature that is strikingly beneficial is Tin Filter, we are able to acquire extremely good quality CT scan of lungs with the dose equivalent to 1 or 2 chest radiographs. This feature is particularly useful for lung cancer screening and the patient requiring regular follow up."



Dr. Jay Vaidya, Horizon Prime hospital, Mumbai

Sub-millimeter collimation

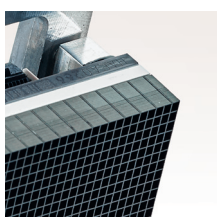

In addition to high power reserves in the generator and tube, SOMATOM go.Now Compact features continuous **0.7 mm collimation across the full detector width**. It therefore achieves uniform scanning over longer ranges without compromising spatial resolution or speed. Moreover, SOMATOM go.Now Compact always provides the thin slice data necessary for flexibility in postprocessing.

Therefore, if you need further details, such as additional orientations or postprocessing tasks, you can retrospectively use inline results for additional reconstructions. You will find these reconstructions ready-to-read directly in your PACS, your film printer, or any other reading environment.

Technical Specifications

Slices / Rotation	96*
Gantry	70 cm
FOV	50 cm (70 cm HD FOV)
Generator	24 kW (60 kW equivalent with SAFIRE)
Tube	3.5 MHU (Upto 8.75 MHU equivalent with SAFIRE)
Tube Cooling Rate	915 kHU/min
Detector	STELLAR with 3D Anti Scatter Collimator
Minimum Slice Thickness	0.6 mm
Iterative Reconstruction	SAFIRE
Workflow	Mobile tablet based remote operation with inline results - Scan&GO, Check&GO, Recon&GO, CTView&GO



* Based on 0.1mm/slice increment reconstruction



"SOMATOM go.Now with compact version is able to scan full body - not only the body trauma scan, but even for whole body angio - thanks to latest generation of X-ray tube which provides uninterrupted clinical capabilities that include multi phasic studies and long range angiographies at lowest possible contrast and patient dose. I am a big fan of integrated colorful "Halo" light which increases patient comfort and especially useful while performing pediatric scanning without any sedation. New software with ease of interface like spine labelling and tablet-based acquisition increased the flexibility and throughput with accuracy. My referrals are higher due to superior image quality and inline reconstruction available in the scanner. Wider field of view enables us to scan traumatic patients and uncooperative patients without any compromise.

Thanks to Siemens Healthineers for creating Artificial intelligence enabled CT scanner and enabling high end technology within the reach of everyone"



Dr. G.S. Bholla, Super Scans, Jalandhar

Siemens Healthcare Private Limited
Unit No. 9A, 9th Floor, North Tower
Godrej One, Pirojshanagar
Eastern Express Highway
Vikhroli East, Mumbai - 400079, India
Tel.: +91 22 3370 0600
E-mail: hc_contact.india@siemens-healthineers.com
Helpline for business queries: 1800-258-5828
CIN: U74999MH2015PTC264859

SHSIN-DI-CT01-V4-112020
(This replaces SHSIN-DI-CT01-V3-042020)

Product upgradation is a continuous process. Hence, data in this brochure is subject to change without prior notice. For the latest information, please get in touch with our Sales Offices.